

Squeeze Moley

User's Manual

MOLEY

1. Product Description

Introducing Squeeze Moley — your go-to tool for true, on-demand freshness.


Perfect for travel, postpartum recovery, post-surgery care, or just daily peace of mind. Quick, hygienic, and refreshingly simple — because Moley believes cleanliness should never require compromise.

2. Product Parameters


Product Name	Squeeze Moley
Model	PB-02
Size	Ø75 mm × 250 mm
Material	EVA + ABS + PC
Capacity	400 ml / 13.5 oz

3. How to Use


Attach the long neck firmly to the bottle base and twist gently to the left. No tools required, no frustration, no flatpack drama.

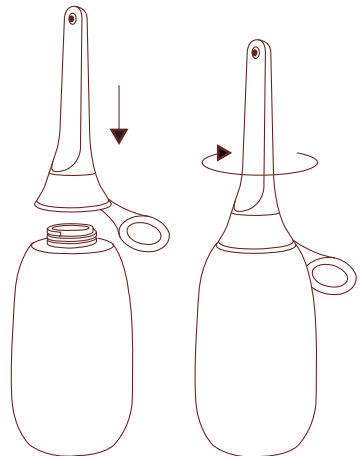

Fill It Up. Two easy filling options: Bend the neck to a 90° angle under the faucet or unscrew, fill directly, and reattach.

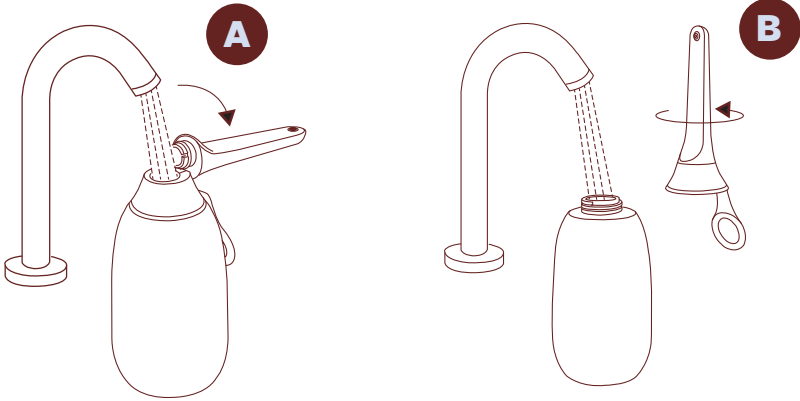


Time to Shine. Flip the Moley upside down, aim for the area that needs the magic, and squeeze to release a smooth, refreshing rinse. More pressure? Squeeze harder. Less? Go gentle. You're in control of your comfort zone.

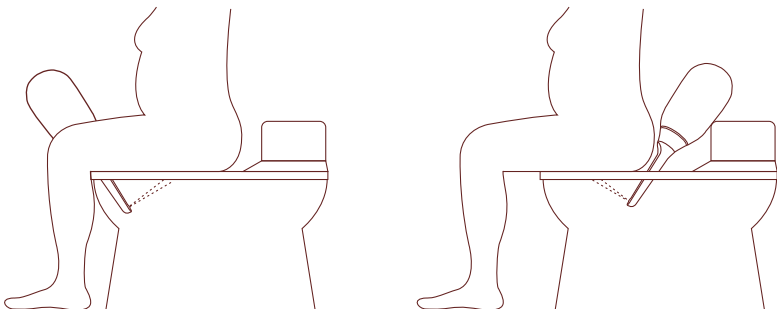


4. Cleaning Methods

First Use: Give Squeeze Moley a rinse with clean water below 60°C / 140°F. Think “warm tea,” not “lava”.

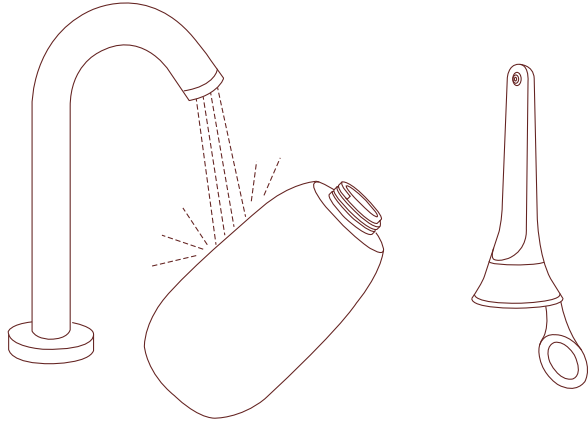

Routine Cleaning: A little mild detergent and some warm water go a long way. Rinse thoroughly, let dry — Moley likes to stay squeaky clean too.

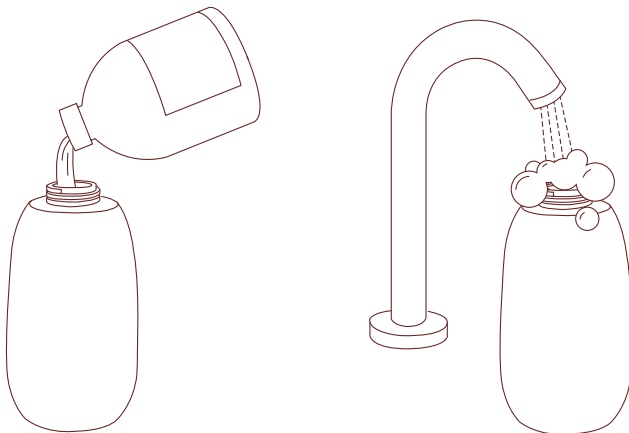

5. Packing List

Everything you need to live fresh and stay confident:




Squeeze Moley x1
The star of the show



Hook x1
Handy little hanger
for easy storage

Silicone ring x1
A snug seal that
keeps it leak-free

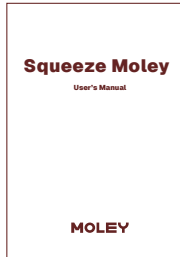

Instruction x1
We recommend
to read it
thoroughly

6. Warnings

Designed for external use only — keep it classy.
The included hook sticks best on clean, dry tiles. Wipe
before you stick.

Always use clean water. Dirty water defeats the entire purpose. Avoid temperatures above 60°C (140°F). After use, empty remaining water and air-dry before storing. Trust us — freshness deserves follow-through.

7. Customer Service

Questions? Enthusiastic testimonials? Existential musings on cleanliness? We're listening. Email us: hello@getmoley.com

8. Manufacturer Information

Shenzhen Xiaohongbao Technology Co., Ltd.
Section A, Yahua Industrial Park,
No. 24 Shapu Guangming Road, Songgang Street,
Shenzhen, Guangdong, China
Model: PB-02 RD EN V03

Moley OÜ / reg code 7381710
Merikotka 37, Kuressaare, Estonia, 93810